

FEGUPOL TI-99

Полировочная паста для титан
Polish for Titanium alloys

ПРОДУКТ ПРЕДНАЗНАЧЕН ТОЛЬКО ДЛЯ
СТОМАТОЛОГИЧЕСКОГО ПРОФЕССИОНАЛЬНОГО
ПОЛЬЗОВАНИЯ

ONLY FOR DENTAL USE BY QUALIFIED PERSONNEL



D-74722 Buchen-Hettingen



Feguramed GmbH
Jahnstr. 2
74722 Buchen (Odenw.)Germany
Te . +49 (0)6281/5227-0 -15
www.feguramed.com

MADE IN GERMANY

Информация о применении

Русский

Показания:

Полировочная паста для титан

Обработка:

1. Предварительная полировка:

1. или 2. режим работы на полировочном моторе с пастой.

Мы рекомендуем щетки (Ø 40 – 80 мм) 3 или 4 ряда с или без вставок из волокон рами.

2. Зеркальный блеск:

Полировочные круги из волокон рами или из шерсти (Ø 40 – 80 мм) во 2. режиме работы



Хранение:

Хранить при температуре не выше 25 °C. Избегать попадания прямого солнечного света. После применения ёмкости следует всегда закрывать и хранить в недоступном для детей месте.

Продажная единица

REF

8079 Fegupol TI-99 250 грамм

Instructions for use

English

Indication:

Polishing paste for Titanium alloys

Technique:


1. Pre-polishing:

With polishing paste at the dental lab lathe (1st or 2nd gear).

We highly recommend brushes (Ø 40 – 80 mm) with multiplerows including black bristleand cotton.

2. High-lustre surface polishing:

With cotton maps (Ø 40 – 80 mm) at the dental lab lathe (2nd gear).

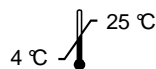

Storage:

Don't store above 25 °C. Keep away from sunlight. Keep locked up and out of reach of children. Keep container tightly closed.

Presentation:

REF

8079 Fegupol TI-99 250 g